



# General Guidelines For Use

Please follow local healthcare policy for personal protective equipment, hand hygiene & waste disposal guidelines.



**1.** Open the device by pulling away the peelable lid.



**2.** Insert the needleless connector fully into the cavity.



**3.** Twist back and forth in semi-circular movements for at least 30 seconds.



**4.** Once disinfected, allow the needleless access site to air dry for 30 seconds.

Always refer to manufacturer's instructions prior to use.

**Find out more about how our Layered Approach to Infection Prevention can help you.**

PDI EMEA, Pywell Road, Willowbrook Est.  
Industrial Estate, Corby - Northants, NN17 5XJ, UK

+44 (0) 8081 697 945

contact@pdi-emea.com

wearepdi-intl.com

PDI International

PDI\_Intl

Space for hospital logo