

# General Guidelines For Use


Please follow local healthcare policy for personal protective equipment, hand hygiene & waste disposal guidelines.



**1** If the surface is visibly soiled, clean prior to disinfecting following an S-shaped motion. Additional wipes may be needed for cleaning heavily soiled surfaces. Use a fresh wipe for each new surface.



**2** Using a fresh wipe, disinfect the surface using an S-shaped motion from top to bottom avoiding cross-contamination.

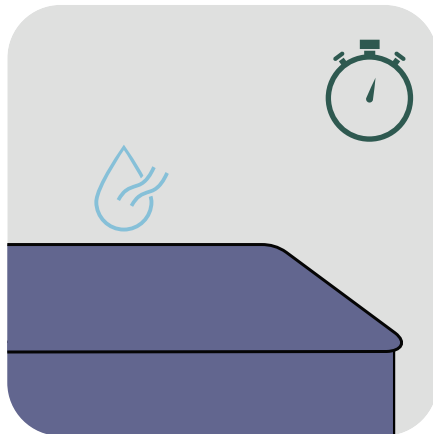

**3** Ensure the surface remains visibly wet for the required contact time, use an additional wipe if needed. Allow surface to air dry. Refer to efficacy table on packaging for contact times.



**4** Dispose of wipe after single use.  
DO NOT FLUSH - DO NOT MACERATE.

Always refer to manufacturer's instructions prior to use.

**Find out more about how our Layered Approach to Infection Prevention can help you.**

PDI EMEA, Pywell Road, Willowbrook Est.  
Industrial Estate, Corby - Northants, NN17 5XJ, UK

+44 (0) 8081 697 945

contact@pdi-emea.com

wearepdi-intl.com

PDI International

PDI\_Intl

Space for hospital logo

**PDI**  
BE THE DIFFERENCE™