



# General Guidelines For Use

## PT1 Human Hygiene Biocide.

Please follow local healthcare policy for personal protective equipment, hand hygiene & waste disposal guidelines.



**1.** Safely remove the applicator from the packaging without any external contact, assisting aseptic non-touch technique (ANTT) compliance.



**2.** Firmly press the dispenser to release the liquid from the chamber onto the sponge.



**3.** Holding the applicator upright, fully press the sponge onto the skin several times to bring the liquid through the sponge.



**4.** Use the soft all-round sponge to apply the liquid to the desired area of the skin to be disinfected. Allow to air dry.

**Find out more about how our Layered Approach to Infection Prevention can help you.**

PDI EMEA, Pywell Road, Willowbrook Est.  
Industrial Estate, Corby - Northants, NN17 5XJ, UK

+44 (0) 8081 697 945

contact@pdi-emea.com

wearepdi-intl.com

PDI International

PDI\_Intl

Space for hospital logo